


AirTouch N30i

Nasal mask

Freedom

The ResMed AirTouch™ N30i features a top-of-head tube design and the new ComfiSoft™ silicone-coated fabric cushion combined with a frame that's fully wrapped in soft, breathable, moisture-wicking fabric.

Systems and Components

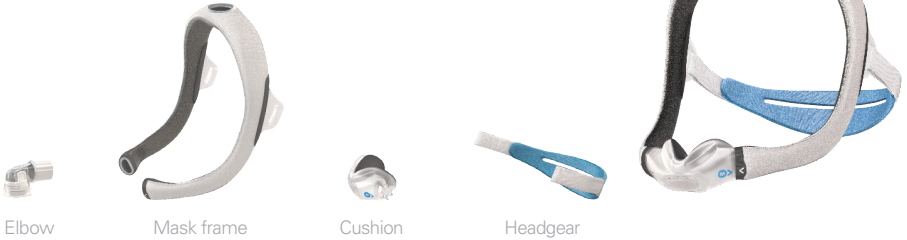

How to fit the AirTouch N30i



Pull the frame and headgear over the head so the elbow sits on top and the headgear is positioned behind the head. Place the cushion under the nose.



Tighten or loosen the headgear until the cushion fits comfortably. Feed longer hair through the back of the frame and over the top of the headgear, if needed.



Squeeze the side buttons on the elbow and detach it from the frame. Connect the air tubing from the device to the elbow, then reattach the elbow to the frame.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule†
Complete mask system (includes frame, cushion and headgear) <ul style="list-style-type: none"> • Starter pack, small—small wide/medium/large cushions • Starter pack, standard—small wide/medium/large cushions • Small frame—small wide/medium/large cushion • Standard frame—small wide/medium/large cushion 	62301 62300 62313/62314/62315 62310/62311/62312	A7034 + A7035	—
Frame system (includes frame and cushion) <ul style="list-style-type: none"> • Small frame—small wide/medium/large • Standard frame—small wide/medium/large 	62319/62320/62321 62316/62317/62318	A7034	Every 3 months
Cushion <ul style="list-style-type: none"> • Small wide • Medium • Large 	62330 62331 62332	A7032	Every 2 weeks
Headgear—Standard	63814	A7035	Every 6 months

† Replacement cycle may vary depending on patient's insurance coverage.

Distributed by ResMed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA (858) 836-5000 or (800) 424-0737 (toll-free). See ResMed.com for other ResMed locations worldwide. [trademarked product names] are trademarks and/or registered trademarks of the ResMed family of companies. Specifications may change without notice. For patent and other intellectual property information, see ResMed.com/ip.

© 2024 ResMed. 10114252/1 2024-11

